




# ODALYS : Addendum to the Surgical Technique


§6&7 of the Odalys surgical technique : ODALYS\_DEG\_TO\_EN\_20121220  
for use of the SD0130 screw holder

## 6 Grasping of the pedicular screw

Assemble the ratchetable handle on the SD0130 screw holder  
Non-sterile implants: choose the appropriate screw on the rack of implants  
Sterile implants: open the box of the selected screw.

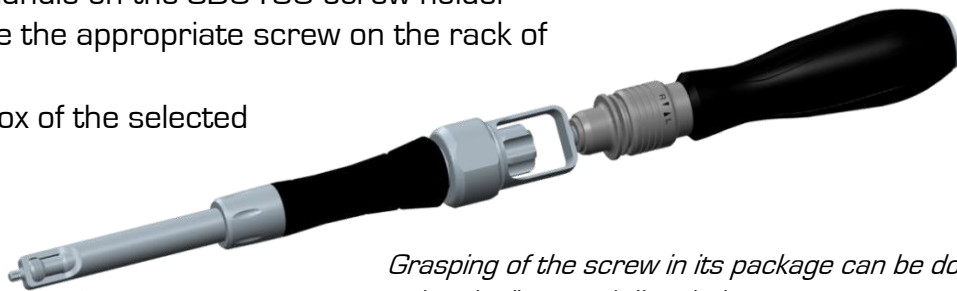


*Grasping of the screw in its package can be done using the "no touch" technique.*

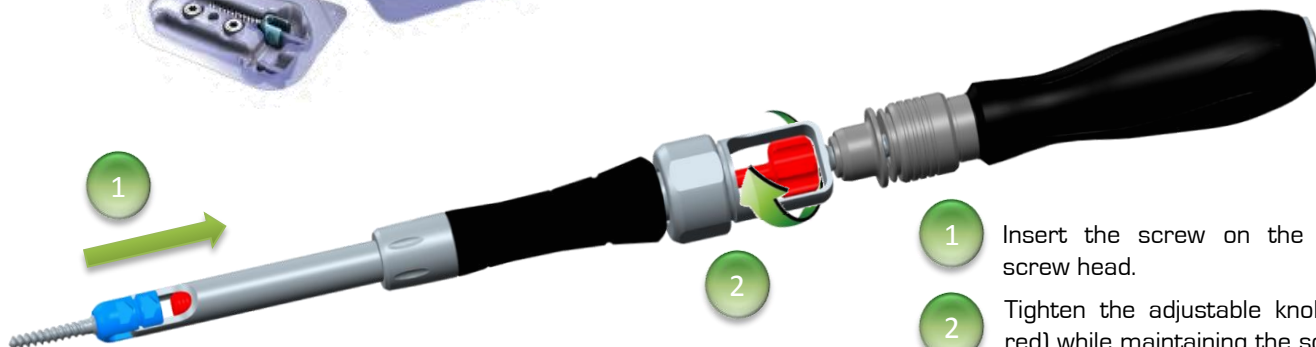

- 1 Insert the screw on the torx screw head.
- 2 Tighten the adjustable knob (in red) while maintaining the screw in the torx imprint.

## 7 Tightening of the pedicular screw

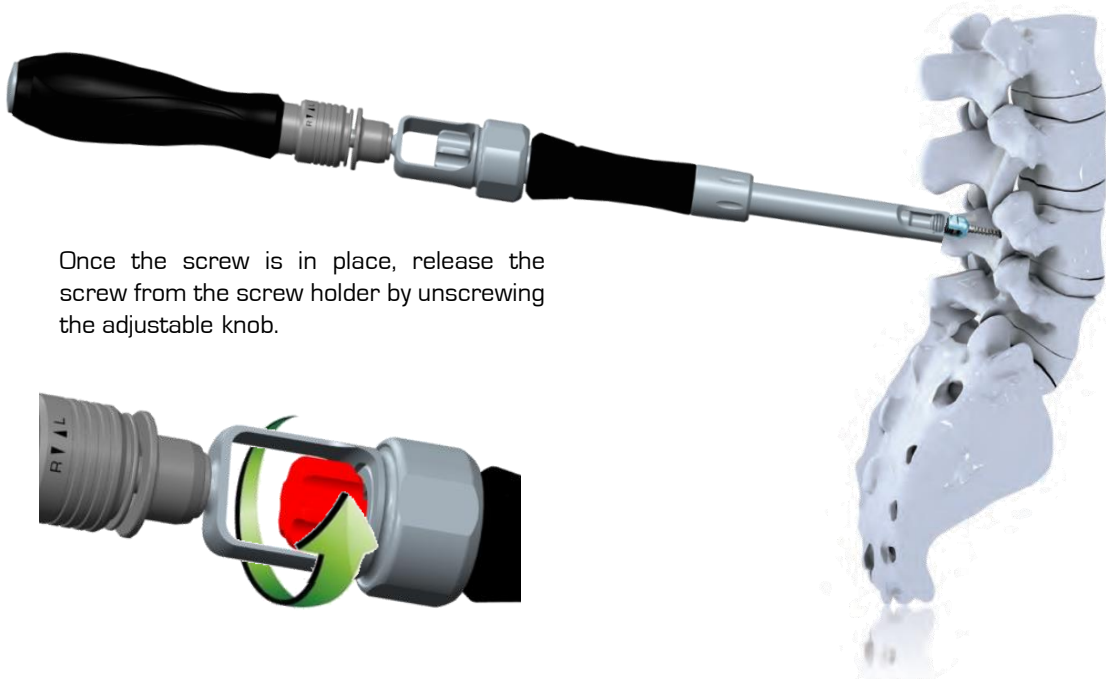

Once the screw is in place, release the screw from the screw holder by unscrewing the adjustable knob.



17/07/2014 Non-contractual images. This document is the property of Kisco International